

ASSEMBLY INSTRUCTIONS

HALLIE LIFT CHAIR



Hallie Recliner Chair with Air Massage & Power Lift

⚠ The components are heavy, assembly should be performed by two people.

If you have any difficulty with assembly or operation, please contact the store of purchase in the first instance.

Please read the below safety & electrical instructions before operating

Safety Instructions


- This product is not intended for use by persons (including children) with reduced physical, sensory or mental capabilities, or lack of experience and knowledge, unless they have been given supervision or instruction concerning use of the product by a person responsible for their safety.
- Ensure there is no person, animal or item obstructing the area under the footrest before operating this product.
- Improper use or installation may result in the risk of fire, electric shock and/or injury. We assume no liability for any eventual damages caused by improper use or inappropriate handling, or by any other non-compliance with the instructions in this manual.
- Children should be supervised to ensure that they do not play with the product.
- This product contains moving parts. Keep fingers, hair and body parts well clear from moving parts. To reduce the risk of injury, keep children and pets clear of the product when in use.
- Do not sit or place additional weight on the arms, back or open footrest.
- This product is very heavy. Only move this product by lifting the main body. Never drag by the footrest, back or arm rest.
- Return this product to the full upright position to stand safely and prevent a tripping hazard.
- Do not operate this product if damage is visible in any form.
- Do not modify or use unauthorised substitute parts.
- This product has been designed and intended for indoor residential use only. It is not suitable for commercial or outdoor use.
- This product is not intended to be used as part of a health care plan. Consult your medical professional before operating if you are taking medication or have physical limitations.

Electrical Instructions

- Never force the back or footrest to close faster than the electric motor.
- Before commencing cleaning or maintenance, ensure this product is disconnected from the power outlet and the back is in the full upright position.
- Disconnect this product from the power outlet before moving.
- Cease operation and disconnect from the power outlet if you ever detect a burning odour or smoke coming from the product.
- Do not operate this product if liquid spillage occurs on or around it. Disconnect from the power outlet and allow product to completely dry before reconnecting.
- Do not operate the power functions or charge station with wet hands.
- Check the condition of the power cord, electrical cords, transformer, plug and connector on a monthly basis. Do not operate if wear is visible.
- Ensure the power cord, electrical cords and transformer are clear of moving parts and not trapped underneath the base.
- Position the power cord to minimise a tripping hazard.
- Never attempt to open or disassemble electrical components.
- We recommend the use of a surge protector (not included).

Synargy is a registered responsible supplier on ERAC. Electrical components within our products have been tested to comply with legal requirements imposed for electrical safety and EMC.

Rear view of recliner chair base with backrest placed on top.

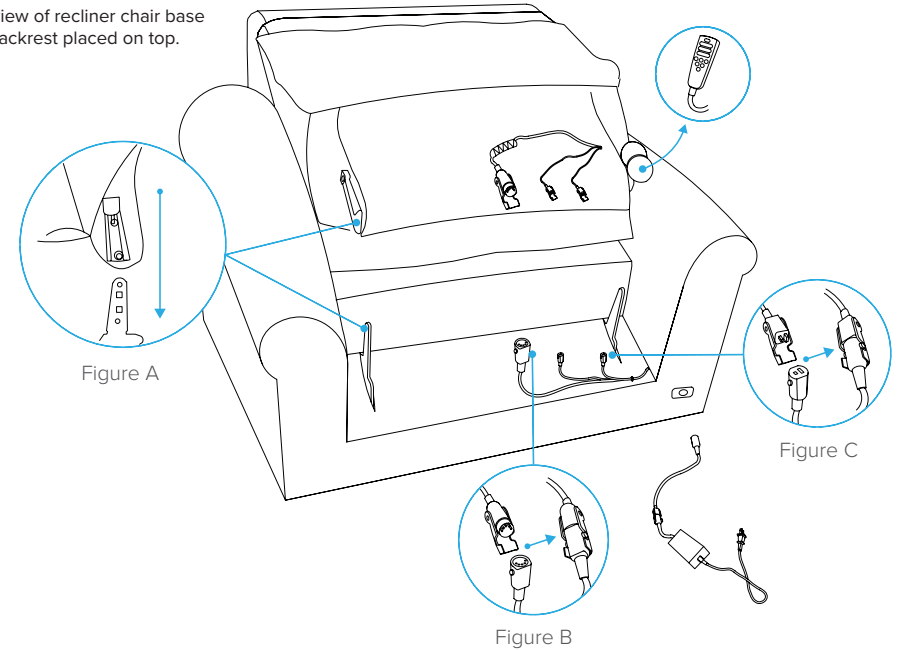

Included Components:

Part	Description	
a	back rest	
b	back rest wings (pair)	
c	seat base	
d	power supply	

Step 1

Remove the power supply from the metal frame on the underside of the chair. Remove packaging from handheld remote control. Place the remote control on the seat base, ensuring the cable will not be in the way during Step 2.

Step 2

Lift the back rest to align the metal receivers with the metal posts of the chair base.

Allow the posts to slide into the receivers (Figure A), and press down firmly on the top of the backrest until a click is heard on each side.

Step 3

Locate the power cables on the back rest, and connect to their respective plugs on the chair base (Figures B and C).

Step 4

Replace the fabric back cover and fasten with velcro on the underside of the chair base. Slide the back rest wings onto each side of the back rest

Step 5

Plug the power supply into the outlet located at the bottom right of the chair base.

Remote Control Operation

